



INSTRUCTION, WL-7

RETRACTABLE SAFETY BELT KIT INSTALLATION

KIT P/N 267493-01

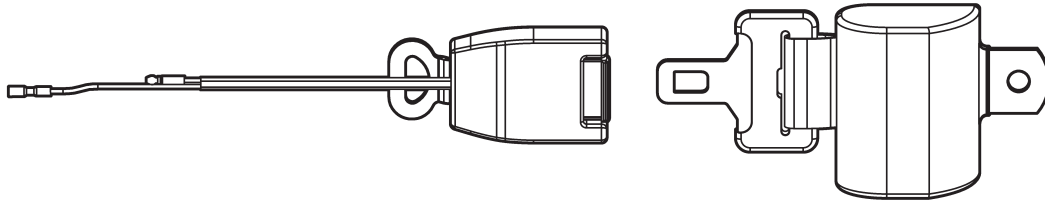

RETRACTABLE SAFETY BELT ASSEMBLY
P/N 267521-01
QTY. 1

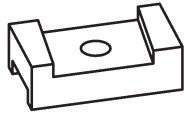

SAFETY BELT MOUNT BRACKET
P/N 267474-01
QTY. 1



CAP SCREW, 5/16"-18 X
1-1/4" LG, GRADE 8
P/N 900009-5
QTY. 1



CAP SCREW, 5/16"-18 X
3/4" LG, GRADE 8
P/N 900009-3
QTY. 2



FLAT WASHER, NYLON
1 1/32" ID X 3/4" OD
P/N 903402-03
QTY. 1




FLAT WASHER
5/16", 1/32" THK
P/N 902000-7
QTY. 3



LOCK NUT, 5/16"-18
P/N 901001
QTY. 3


1. See the WL7 Operation Manual for operating instructions. Then apply power to the Lift by turning **ON** the switch on the Pump Cover.

2. Next **UNFOLD** the Lift and lower the Platform to ground level (**FIG. 2-1**). Remove power from the Lift by turning **OFF** the switch on the Pump Cover.

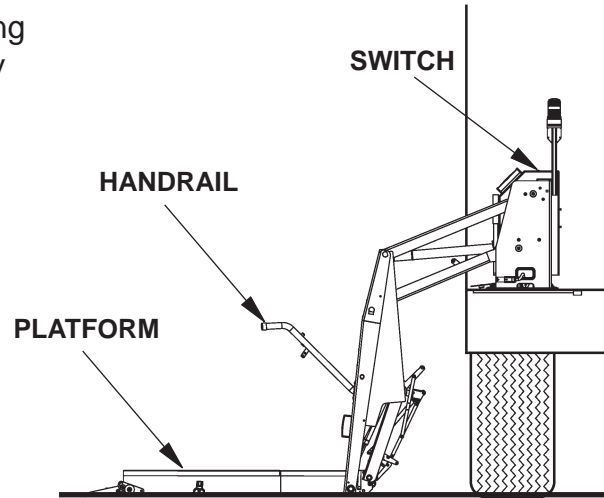

**RH SIDE SHOWN
FIG. 2-1**

3. Remove cover from bracket on Right Hand (RH) side and Left Hand (LH) side Handrails (**FIGS. 2-2A, 2-2B, and 2-2C**).

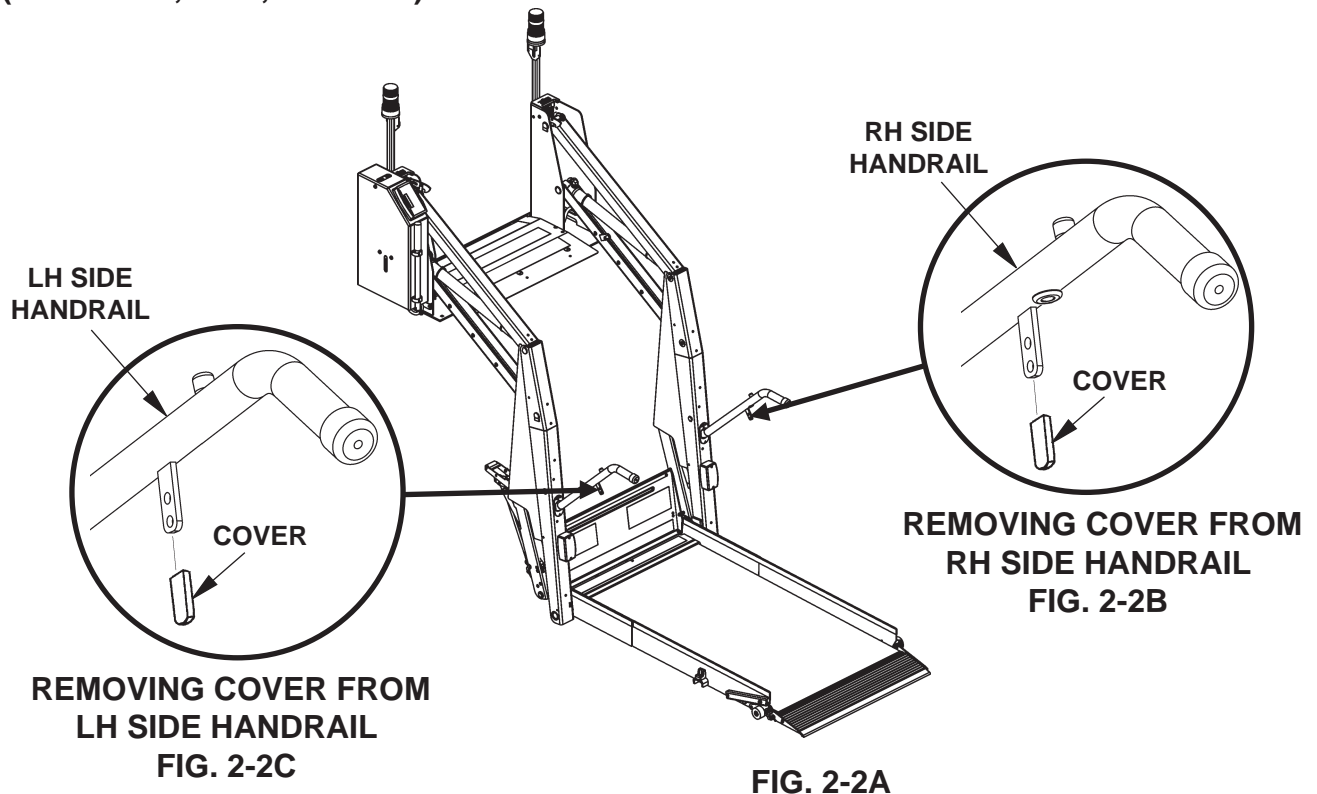

4. Remove rubber grommet from Right Hand (RH) side Handrail (**FIGS. 3-2A and 3-2B**).


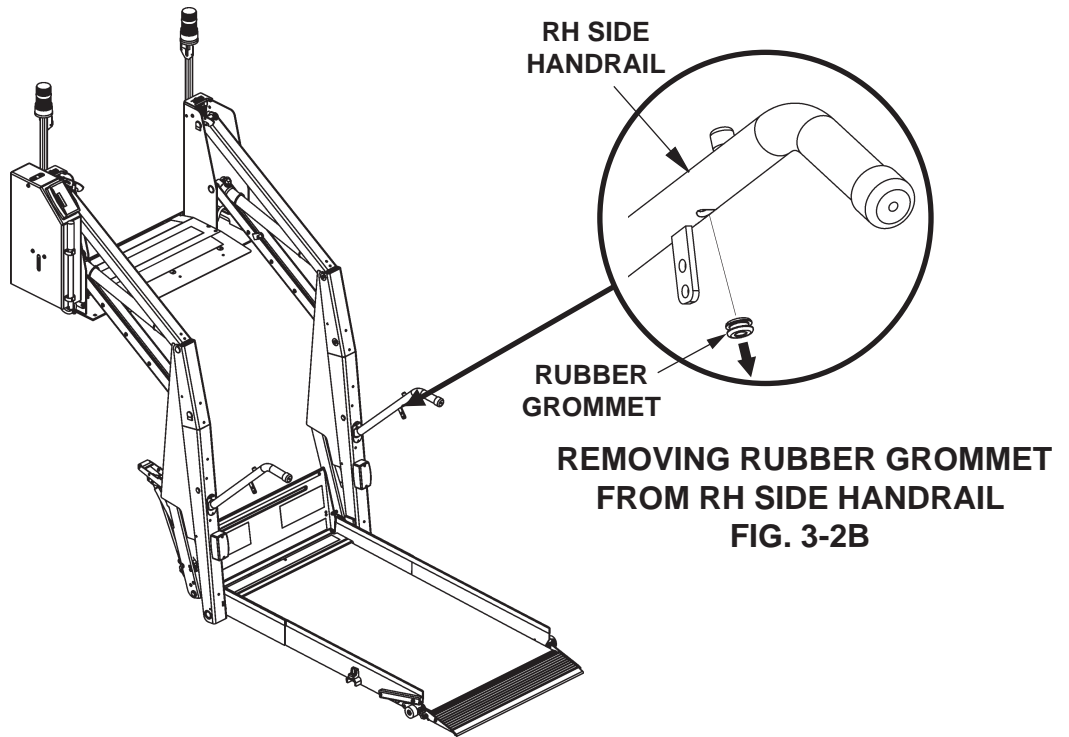



FIG. 3-2A

5. Fish the wiring harness from the RH side Handrail (**FIG. 3-2**) and then feed it through the Rubber Grommet. Reinstall the Rubber Grommet.

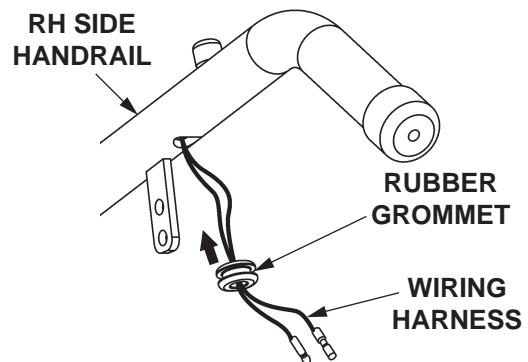


**FISHING WIRING HARNESS FROM RH HANDRAIL
FIG. 3-2**

6. Unbuckle the Safety Belt (Kit item)
(FIG. 4-1).

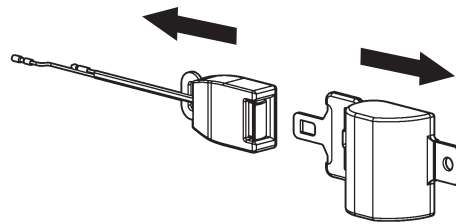


UNBUCKLING SAFETY BELT
FIG. 4-1

7. Bolt the Seat Belt Buckle on the RH Handrail as shown in FIG. 4-2.

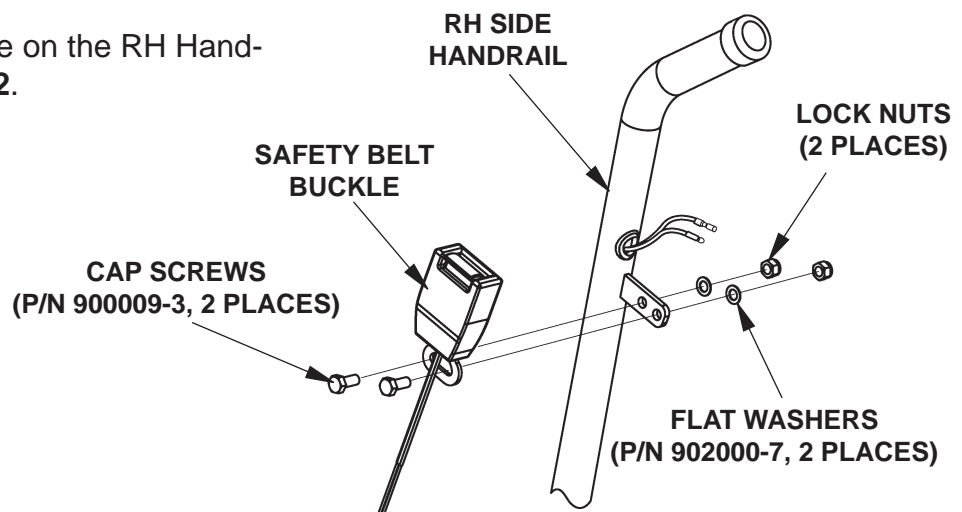


BOLTING SAFETY BELT TO
RH SIDE HANDRAIL
FIG. 4-2

8. Connect the Safety Belt wiring harness to the Handrail wiring harness. Feed the extra wiring back in the Handrail until the loom reaches the grommet. (FIG. 4-3).

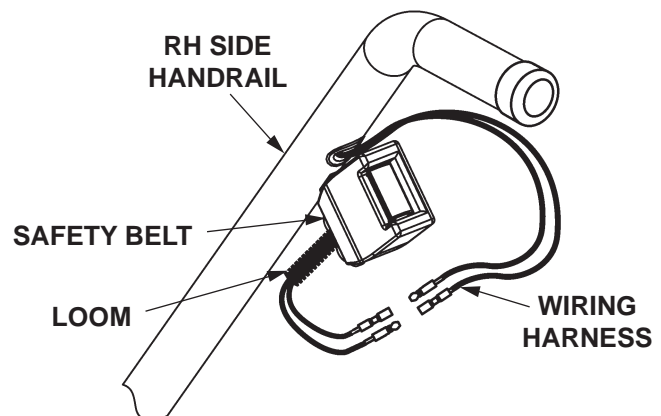


CONNECTING SAFETY BELT WIRING HARNESS
TO HANDRAIL WIRING HARNESS
FIG. 4-3

9. Bolt the Retractable part of Seatbelt Assembly to the LH side Handrail as show in **FIGS. 5-1A and 5-1B.**

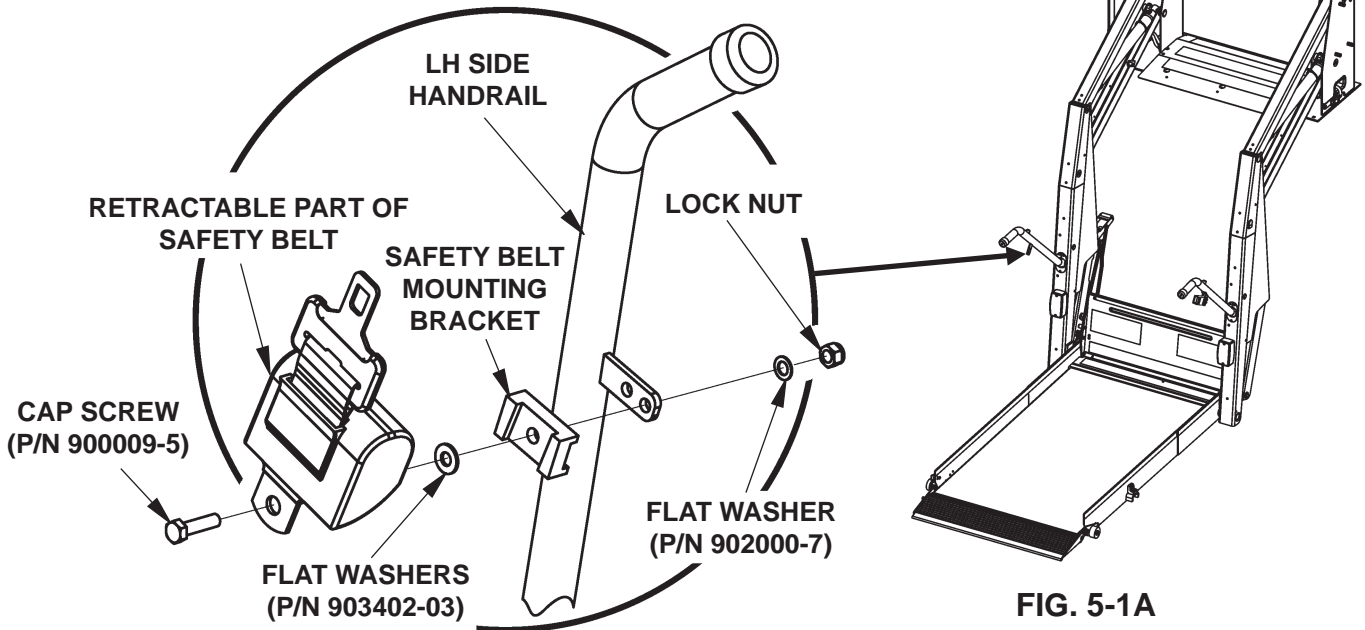


**BOLTING SAFETY BELT TO
LH SIDE HANDRAIL
FIG. 5-1B**

10. Use the Hand Control to raise **UP** the Platform off the ground as shown in **FIG. 5-2.**

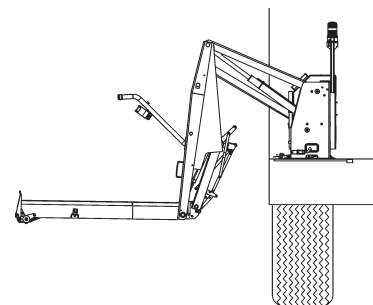


**PLATFORM RAISED
OFF THE GROUND
FIG. 5-2**

11. Unbuckle the Safety Belt (**FIG. 6-1**). Use the Hand Control to raise **UP** the Platform. Platform should not rise and Controller should read **UP SEAT BLT** (**FIG. 6-2**).

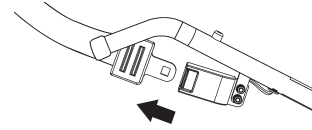


UNBUCKLING SAFETY BELT
FIG. 6-1

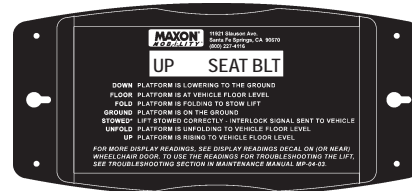


CONTROLLER READING
FIG. 6-2

12. Buckle the Safety Belt (**FIG. 6-3**). Use the Hand Control to raise the Platform to floor level. Platform should raise to floor level (**FIG. 6-4**).

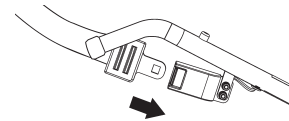

BUCKLING SAFETY BELT
FIG. 6-3

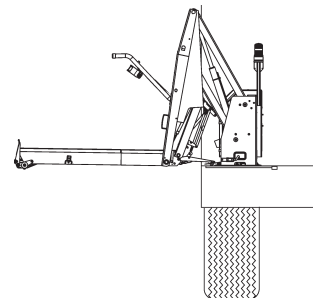

PLATFORM RAISED TO
FLOOR LEVEL
FIG. 6-4

13. Stow the Lift.